PST®
Instrument Trays
Instructions for Use

- REF 1300A, 1301A, 1310A, 1311A, 1320A, 1321A, 1500A, 1501A, 1520A, 1521A, 1522A, 1523A, 1524A, 2341A, 2343A, 2361A, 2363A, 2501A, 2502A, 2503A, 2505A, 2571A, 2572A, 2573A, 2600A, 2601A, 2610A, 2611A, 2700A, 2701A, 3361A, 3363A, 4600A, 4601A, 4700A, 4701A, 4710A, 4711A, 6100A, 6101A, 6120A, 6121A, 6122A, 6123A, 6124A, 6125A, 6130A, 6131A, 6132A, 6133A, 6134A, 6135A, 6140A, 6141A, 6150A, 6151A, 6500A, 6501A, 6502A, 6510A, 6511A, 6512A, 6520A, 6521A, 6522A, 6550A, 6551A, 6552A, 6560A, 6561A, 6562A, 1380M, 1381M.
- Designed to encase instruments for sterilization in steam.
- An appropriate sterile barrier will be required to maintain sterility.
- Use of the PST Instrument Trays must be in accordance with the operating instruction supplied with your particular sterilization equipment and in keeping with your particular sterilization equipment and in keeping with your hospital policy on sterilization validation. Several organizations (e.g., AAMI, AORN and ISO) publish guidance information on sterilization usage should you need additional information.
- Periodic cleaning of the trays after sterilization process is recommended to prolong usable life.
- **Do not use if damaged**

Accessories for PST® Instrument Trays

PST Instrument Tray Mats

PST Instrument Tray Holders

PRECAUTIONS/INSTRUCTIONS:

1. Prior to initial use, clean trays per instructions provided.
2. Inspect trays for damage, such as sharp edges that could cause a tear in the sterile wrap.
3. DO NOT drop sterilization tray.
4. DO NOT use with nonabsorbent tray liners or other accessories not provided by PST. Nonabsorbent tray liners can cause condensate to pool.
5. Trays should be processed according to the IFU of the sterilization packaging manufacturer, the enclosed device(s) manufacturer, and the sterilizer manufacturer.
6. After the sterilization cycle is complete and sterilization carriage is removed, all trays should be allowed to cool to room temperature before handling and dispensing. The amount of time required depends on load content and ambient conditions (temperature and humidity). The potential for condensation may increase if the case is not allowed to cool properly.
7. Gross soiling should be reduced by wiping down the surfaces of the tray prior to transportation.
8. Only use PST supplied accessories with PST Instrument trays. Blocking tray vent holes may cause sterilization to be ineffective.

WARNINGS:

1. DO NOT use with sterilization methods other than those indicated for use. DO NOT use with gamma sterilization.
2. DO NOT use disinfecting solutions to sterilize the trays or accessories.
3. DO NOT stack trays on top of one another. Be sure that ventilation holes are not obstructed. For effective steam sterilization, trays must have adequate steam penetration through all surfaces. They must be placed upright on shelves for proper ventilation. Condensation can pool on non-absorbent surfaces.
4. DO NOT place trays on their sides or at vertical angles to ensure proper drainage.
5. DO NOT overload trays. Overloading can inhibit steam flow, cause excessive drying times, and make trays too heavy to safely handle.
6. USE ONLY with an approved sterile wrap indicated for use with sterilization method. Trays are **not** designed to maintain sterility.
7. Any serious incident that has occurred related to PST Trays should be reported to PST and a local member state competent authority.
8. DO NOT expose trays or accessories to temperatures above 350F

Cleaning Instructions:

Follow universal precautions for protective apparel when handling and cleaning contaminated equipment.

Manual Cleaning is permissible but not recommended:

1. Remove all product and PST tray accessories from trays prior to cleaning.
2. Wipe the tray and accessories with a clean, soft, non-abrasive brush dampened with a neutral pH-balanced detergent.
3. Thoroughly rinse all surfaces of residual detergent and soil.
4. Inspect trays and accessories to ensure that there is no sign of visible debris left on tray or accessory surfaces including tray perforations.
5. Repeat steps 3 to 5 as needed until the tray is visually clean.
6. Dry surfaces using a clean, non-linting cloth.

Machine Cleaning:

- Machine cleaning is recommended in a washer disinfector conforming to ISO 15883- 1 & 2.
- Products should be positioned in the washer to allow maximum water penetration and drainage. Lids should be open/removed.
- Products can be cleaned with neutral pH detergents. Consult the IFU of the detergent manufacturer for material compatibility of the detergent(s) used.
- Final rinse, AAMI TIR34 Critical Water is recommended.
- The standard program to include:
 1. Pre-wash with cold water <38°C rinse for a minimum of 2 minutes.
 2. Washing cycle with enzymatic detergent at temperature recommended by the detergent manufacturer for a minimum of 4 minutes.
 3. Washing cycle with neutral pH detergent at the temperature recommended by the detergent manufacturer for a minimum of 2 minutes.
 4. Final rinse cycle for a minimum of 1 minute (preferably with AAMI TIR 34 Critical Water.)
 5. Thermal disinfection of up to 93°C in compliance with the washer manufacturer recommendations for time and temperature.
 6. Drying cycle at temperature not to exceed 115°C.
 7. Caution when unloading machine as products will be hot.
 8. Visual inspection is required to ensure complete removal of soil.
 9. If product still shows soil, repeat program.
 10. Drying may be accomplished during the dry cycle of the automated washer and/or with the use of a non-linting cloth.

Disposal:

Follow standard hospital policy for disposing of medical device waste. Prior to disposal, disinfect trays/accessories and dispose of as biohazardous material.

 **Plastic Sterilizing Tray Corporation**
1520 East Street, Pittsfield, MA 01201, USA

Authorized EC Representative:
MediMark Europe
Siege Social 11
rue Emile Zola-BP 2332
38033 Grenoble Cedex 2-France
Tel: : +33 (0)4 76 86 43 22
Fax: +33 (0)4 76 17 19 82
E-Mail: medimark@easynet.fr
3G5M Rev 01

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